




# Comfort Shield®

8 barrier cloths/package  
with Peri Check™ Guide

## Warmer Instructions

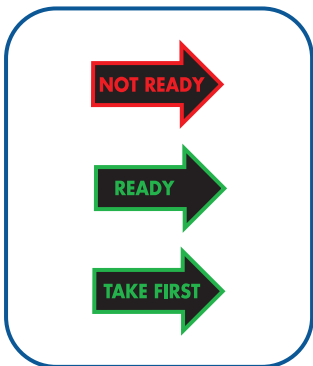

**1** To stock warmer, place one package in each slot.

Red **"Not Ready"** indicators will light up.

Within 2 hours, green **"Ready"** indicators will light up. These products are ready for use.

If green **"Take First"** indicators are flashing, use those packages first.

**2** Remove package for use when **"Ready."**

**3** Remove wet and soiled garment.



**4** Peel label back to remove barrier cloths. Remove excess fecal matter using briefs, chux, underpad, Comfort Shield barrier cloth or other appropriate method per hospital protocol.

**5** Cleanse soiled area using a fresh cloth. A barrier will be left on the skin. Use as often as necessary, especially if prolonged exposure to wetness is anticipated.



**6** Pull off Peri Check Guide if needed.

**7** Fill out info on back of Peri Check Guide and follow your reporting protocol.



*See product package for complete information.*

**CAUTION:** It is very important to check the temperature of the barrier cloths. If there is any indication that barrier cloths are excessively hot, do not use. Remember, gloves diminish your sensitivity to heat. During cleansing, communicate with your patient to make sure each barrier cloth is at a comfortable temperature. Closely monitor heat level with infants, unresponsive or skin sensitive individuals.

**WARNING:** For external perineal use only. Avoid contact with the eyes. If condition worsens, or does not improve within 7 days, consult a doctor. Keep out of reach of children. If swallowed, get medical help or contact a Poison Control Center right away.

**SAGE®**  
PRODUCTS INC  
[www.sageproducts.com](http://www.sageproducts.com)  
800-323-2220