



# Comfort Rinse-Free Shampoo Cap<sup>®</sup>

## Microwave Instructions



**1** Before warming, tear open package without removing cap.

**2** Warm package for **no more than 30 seconds.**

*\* Microwave wattage varies; adjust heating time accordingly.*



**3** Place cap on head. For long hair, place hair on top of head to get all of it into cap. Gently massage cap to saturate hair. Some situations may require cap to be left on head to saturate tangled areas. Remove cap. Comb through. Towel dry.

**Amount of time massaging cap depends on quantity of hair.**

- **Short/Fine Hair: Approximately 1 to 2 minutes**
- **Long/Thick Hair: Approximately 4 to 5 minutes**

*See product package for complete information.*

**CAUTION:** If there is any indication cap is excessively hot, do not use. Remember, gloves diminish your sensitivity to heat. Closely monitor heat levels with unresponsive or skin sensitive individuals.

**NOTE:** Washcloths may be necessary to remove excessive amounts of blood, gels or dirt before using product.

**SAGE**<sup>®</sup>  
PRODUCTS INC  
[www.sageproducts.com](http://www.sageproducts.com)  
800-323-2220