



Comfort Rinse-Free Shampoo Cap

warmer instructions



- 1 To stock warmer, place one package in each slot.

Red **"Not Ready"** indicators will light up.

Within 2 hours, green **"Ready"** indicators will light up. These products are ready for use.

If green **"Take First"** indicators are flashing, use those packages first.

- 2 Remove package for use when **"Ready."**

- 3 Tear open package.

- 4 Place cap on head. For long hair, place hair on top of head to get all of it into cap. Gently massage cap to saturate hair. Some situations may require cap to be left on head to saturate tangled areas. Remove cap. Comb through. Towel dry.




Amount of time massaging cap depends on quantity of hair.

- **Short/Fine Hair: Approximately 1 to 2 minutes**
- **Long/Thick Hair: Approximately 4 to 5 minutes**

See product package for complete information.

CAUTION: If there is any indication cap is excessively hot, do not use. Remember, gloves diminish your sensitivity to heat. Closely monitor heat levels with unresponsive or skin sensitive individuals.

NOTE: Washcloths may be necessary to remove excessive amounts of blood, gels or dirt before using product.

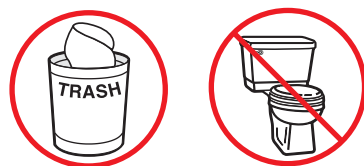

DO NOT FLUSH WASHCLOTHS IN TOILET!

SAGE
PRODUCTS INC
www.sageproducts.com
800-323-2220