



Deodorant Comfort Bath[®]

8 washcloths/package
with Skin Check[™] Guide



Microwave Instructions



1 Before warming, peel label back.



2 Warm package for **no more than 60 seconds.*** Reseal label to retain warmth.

* *Microwave wattage varies; adjust heating time accordingly.*



3 To bathe patient, use one washcloth for each area to be cleansed.

4 Pull off Skin Check Guide if needed.

5 Fill out info on back of Skin Check Guide and follow your reporting protocol.

Recommended Bathing Procedure



Use a clean washcloth for each area

FRONT

1. Face, Neck & Chest
2. Right arm/Axilla
3. Left arm/Axilla
4. Perineum

BACK

5. Right leg
6. Left leg
7. Back
8. Buttocks

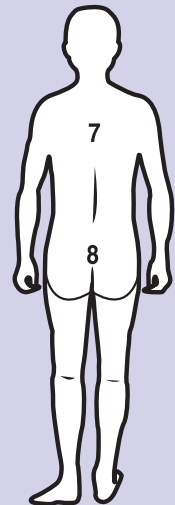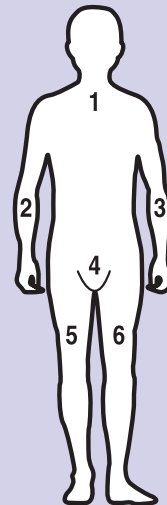

DO NOT FLUSH

DO NOT DISCARD CLOTHS IN TOILET

See product package for complete information.

CAUTION: It is very important to check the temperature of the washcloths. If there is any indication that washcloths are excessively hot, do not use. Remember, gloves diminish your sensitivity to heat. During bathing, communicate with your patient to make sure each washcloth is at a comfortable temperature. Closely monitor heat level with infants, unresponsive or skin sensitive individuals.

SAGE[®]
PRODUCTS INC

www.sageproducts.com

800-323-2220